DeVilbiss® PulmoNeb™ Compressor/Nebulizer

Models 3655DX, 3655TX, 3655IX,
3655KX & 3655PX

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Instruction Guide

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Guide d'Instructions

CAUTION— Federal (U.S.A.) law restricts this device to sale by, or on the order of a physician.

Made in China. Latex Free.

Models 3655DX & 3655TX Entela
Certified to UL1431, CAN/CSA C22.2
No. 601.1 and IEC 60601-1

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Table of Contents

TABLE OF CONTENTS

Important Safeguards	4
IEC Symbols	4
Important Safeguards	4
Introduction.....	6
Statement Of Intended Use	6
What Are Compressor Nebulizer Systems Used For?.....	6
How Does A Compressor Nebulizer Work?	6
How Do I Take A Treatment?.....	7
Important Parts.....	8
Important Parts Of Your Devilbiss 3655x Series Compressor	8
Important Parts Of Your Disposable Nebulizer	9
Important Parts Of Your Reusable Nebulizer.....	9
Operation.....	10
How To Operate Your Compressor	10
Cleaning & Maintenance	12
Nebulizer Cleaning.....	12
Compressor Cleaning	13
Filter Change.....	13
Maintenance	13
Specifications	14
Limited Warranty	15

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ON

OFF

Attention, consult
instruction guide.

Alternating Current

Type BF Applied Part

Double Insulated

This device contains
electrical and/or
electronic equipment
that must be recycled
per EC Directive
2002/96/EC - Waste
Electrical and Elec-
tronic Equipment
(WEEE)

IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be followed. Read all instructions before using. Important information is highlighted by these terms:

DANGER– Urgent safety information for hazards that will cause serious injury or death.

WARNING– Important safety information for hazards that might cause serious injury.

CAUTION– Information for preventing damage to the product.

NOTE– Information to which you should pay special attention.

READ ALL INSTRUCTIONS BEFORE USING.

! DANGER!

To reduce the risk of electrocution:

1. Always unplug this product immediately after using.
2. Do not use while bathing.
3. Do not place or store product where it can fall or be pulled into a tub or sink.
4. Do not place in or drop into water or other liquid.
5. Do not reach for a product that has fallen into water. Unplug immediately.

! WARNING!

To reduce the risk of burns, electrocution, fire or injury to persons:

1. The product should never be left unattended when plugged in.
2. Close supervision is necessary when this product is used by, on, or near children or invalids.
3. Use this product only for its intended use as described in this guide. Do not use attachments not recommended by the manufacturer.

4. Never operate this product if:
 - a. It has a damaged power cord or plug.
 - b. It is not working properly.
 - c. It has been dropped or damaged.
 - d. It has been dropped into water.

Return the product to an authorized De'Vilbiss service center for examination and repair.

5. Keep the power cord away from heated surfaces.
6. Never block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, and the like.
7. Never use while drowsy or asleep.
8. Never drop or insert any object into any opening or hose.
9. Do not use outdoors. This product is for household use only.
10. Do not use in an oxygen enriched environment.
11. Do not use this product with a DC to AC converter, or with any AC voltage and/or frequency other than specified.
12. Do not operate unit in a backpack or carrying bag.
13. Care should be taken when this product is used in the vicinity of other electronic equipment concerning the possible interference between them. They may need to be moved apart.
14. Use caution when plugging in and unplugging device. Do not allow jewelry or other metal items to contact plug.
15. Warning: This product contains small parts that may represent a choking hazard.

NOTE– This unit employs a temperature and/or current limiting device that will shut off the entire product should a malfunction occur or in the event of product misuse, abuse, or abnormal operation. Contact your qualified De'Vilbiss provider for a replacement unit if necessary.

Some models of this appliance have a polarized plug (one blade is wider than the other). As a safety feature, this plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. Do not attempt to defeat this safety feature. If repair or replacement of the cord or plug is necessary, contact your qualified De'Vilbiss provider.

NOTE– The exterior surfaces of the unit and the air outlet connector may become slightly warm to the touch during use. Warm air can also be felt exiting the vents on the sides of the case of the unit. This is normal.

SAVE THESE INSTRUCTIONS

Your doctor has prescribed a liquid medication to treat your respiratory condition. To best utilize this liquid medication, he has prescribed a DeVilbiss brand compressor nebulizer. Your DeVilbiss compressor nebulizer works to convert the medication to a high-quality mist of fine particles that penetrates deep into the lungs.

Be sure that you read and understand the information in this instruction guide. By following these simple instructions and the advice of your physician, your compressor will become an effective addition to your therapeutic routine.

STATEMENT OF INTENDED USE

The DeVilbiss compressor nebulizer includes an AC powered air compressor that provides a source of compressed air for home health care use. The compressor is used in conjunction with a jet (pneumatic) nebulizer to convert certain inhalable drugs into an aerosol form for inhalation by a patient. The device can be used with adult or pediatric patients.

NOTE– If using a nebulizer other than one distributed by DeVilbiss, verify that the pressure and/or flow characteristics of the nebulizer (while driven by the DeVilbiss compressor) meet the nebulizer manufacturer’s recommendation.

WHAT ARE COMPRESSOR NEBULIZER SYSTEMS USED FOR?

People with breathing problems, such as Asthma, Chronic Obstructive Pulmonary Disease (COPD) or Cystic Fibrosis are often prescribed medicines that minimize or control symptoms of their disease. Compressor Nebulizer Systems are a means of delivering these prescribed medications to the lungs. These medications and the compressor nebulizer systems should be used as directed by a physician to maximize benefits. Compressor Nebulizer systems can be prescribed for use everyday or periodically as symptoms present themselves, such as the springtime when Hay Fever or Allergies are normally present.

HOW DOES A COMPRESSOR NEBULIZER WORK?

Most medicines for use with a compressor nebulizer system are created in a liquid form. This liquid medication is placed in the “cup” of the nebulizer (please refer to this Instruction Guide for more information on the proper use of a nebulizer). The compressor, which is connected to the nebulizer by flexible tubing, generates a specific pressure and flow of air that essentially “drives” the nebulizer. The pressurized air produced by the compressor initiates the nebulizer’s ability to break down the liquid medication into a very fine mist. This mist, often referred to as an aerosol cloud, is then propelled out the nebulizer and inhaled into the lungs where the medicine is readily absorbed. Please consult your prescription for details on the amount of liquid medication that should be placed in the nebulizer. If you should have any questions about your medication or its use, please contact your physician or pharmacist.

HOW DO I TAKE A TREATMENT?

A nebulizer is the optimal delivery method of medication to the lungs. However, some users may find it difficult to use, especially very young children. In this case, an aerosol mask may be used. To ensure that the proper mask is utilized, please consult your home health provider or physician.

The most important aspect of taking a treatment is the patient's breathing pattern while inhaling the medication mist.

It should be slow and deep, all attempts should be made to sit up straight and breathe in deep.

A general rule of thumb is to attempt to breathe in so deeply that it feels as though the lungs are completely full of air and inhaling any additional air is impossible.

If possible, hold your breath unless directed otherwise by your physician and then slowly exhale through the nebulizer mouthpiece or mask. Holding your breath after inhaling the medication allows for optimal absorption of medication in the lungs.


Repeat this breathing pattern for the entire treatment.

The length of a treatment can last anywhere from 5 to 20 minutes, depending on the amount of medication and the performance of your compressor and nebulizer system. When you cannot see any medication in the nebulizer cup and the nebulizer has stopped producing a mist, the treatment has been completed. Turn off your compressor and follow the Cleaning instructions found in this instruction guide.

If you have any additional questions regarding your equipment, medication, or treatment, please consult your physician.


IMPORTANT PARTS OF YOUR DEVILBISS 3655X SERIES COMPRESSOR (Figures A & B)

1. Power switch
2. Filter (non-washable)
3. Air-outlet connector
4. Carrying handle
5. Power cord storage wrap area
6. Nebulizer holder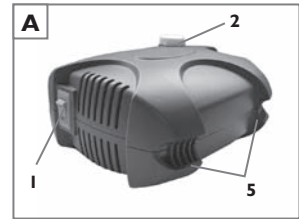

Accessories

3655D-601	Filter (5 pack)
3655D-602	Filter (25 pack)
800D	Reusable Nebulizer (1 each)
4650D-621	Disposable Nebulizer (50 pack)
AAM12	Adult Aerosol Mask (12 pack)
AAM20	Adult Aerosol Mask (50 pack)
PAM10	Pediatric Aerosol Mask (50 pack)
DL1050	Pediatric Dragon Mask (50 pack)

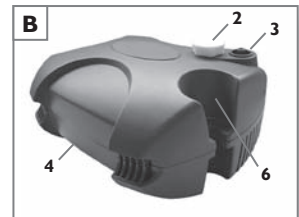
IMPORTANT PARTS OF YOUR DISPOSABLE NEBULIZER

1. Mouthpiece
2. T-piece
3. Cap
4. Baffle
5. Medication Cup
6. Nebulizer Air-Inlet Connector

Nebulizer parts are not sold separately. Additional nebulizers are available through your DeVilbiss provider.

NOTE— Your neb may look different than pictured.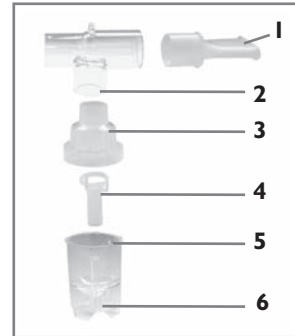


IMPORTANT PARTS OF YOUR REUSABLE NEBULIZER

1. Mouthpiece
2. Cap
3. Baffle
4. Medication Cup
5. Nebulizer Air-Inlet Connector
6. Interrupter Adapter (if applicable - 800G)

Nebulizer parts are not sold separately. Additional nebulizers are available through your DeVilbiss provider.

NOTE— Your neb may look different than pictured.

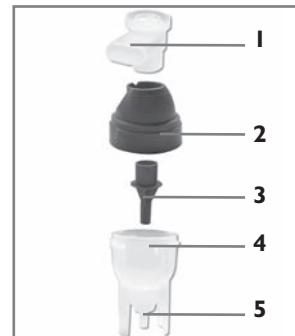HOW TO OPERATE YOUR COMPRESSOR

NOTE– Before initial operation, your nebulizer should be cleaned following cleaning instructions, or as recommended by your physician or DeVilbiss provider.

1. Make sure the power switch is in the “○” position (Fig. 1) and unwrap power cord.
2. Place compressor so that the rubber feet are resting on a level, sturdy surface and upon which the controls can be easily reached when seated. Never block the air openings of the product or place it on a soft surface, such as a patient’s lap or a bed or couch, where the air openings may be blocked.
3. Plug power cord into an appropriate wall outlet.

! DANGER!

DeVilbiss compressor/nebulizer must be operated on the specified power source to avoid risk of electric shock and damage to the compressor.

4. Wash hands.
5. Connect one end of the nebulizer tubing to the compressor air-outlet connector (Fig. B).

NOTE– During high-humidity weather conditions, condensation (water build up) can occur in the nebulizer tubing.

6. Assemble clean nebulizer parts by placing baffle down in medication cup. Add prescribed medication into the medication cup (Fig. 2). Holding cup stationary, screw on nebulizer cap (Fig 3).
7. Assemble mouthpiece and t-piece (if applicable) and insert into the top of the nebulizer cap. If using an aerosol mask, insert the bottom part of the mask directly into the top of the nebulizer cap.
8. Attach tubing to nebulizer air-inlet connector (Fig. 4).
9. Press the power switch “I” (Fig. 1) to start the compressor.
10. Begin treatment by placing the mouthpiece between teeth.

With mouth closed, inhale deeply and slowly through mouth as aerosol begins to flow, then exhale slowly through the mouthpiece (Fig. 5). If treatment needs to be interrupted, simply press power switch “○” and place the nebulizer into the nebulizer holder (Fig. B).

NOTE— Your physician may recommend a specific breathing pattern during treatment. If so, follow his/her recommendations.

11. If an aerosol mask is used, place the mask over mouth and nose (Fig. 6). As aerosol begins to flow, inhale deeply and slowly through mouth, then exhale slowly.
12. When treatment is complete, turn unit off by pressing the power switch to the “○” position. Unplug the unit from the wall power outlet.
13. Cord storage can be done by holding the compressor by its handle and wrapping the cord clockwise around the compressor (Fig. 7).

NEBULIZER CLEANING

All parts of the nebulizer, except tubing, should be cleaned according to the following instructions. Your physician and/or DeVilbiss provider may specify a certain cleaning procedure. If so, follow their recommendations.

WARNING!

To prevent possible risk of infection from contaminated medication, cleaning of the nebulizer is recommended after each aerosol treatment. Disinfecting is recommended once a day.

Clean After Every Use:

1. With power switch in the “○” position, unplug power cord from wall outlet.
2. Disconnect tubing from the air-inlet connector and set aside.
3. Disassemble mouthpiece or mask from cap. Open nebulizer by turning cap counterclockwise and remove baffle.
4. Wash all items, except tubing, in a hot water/dishwashing detergent solution. Rinse under hot tap water for 30 seconds to remove detergent residue. Allow to air dry.
5. For reusable nebulizer only, it can be cleaned daily in dishwasher using top shelf or by following the above cleaning procedure.

CAUTION!

The DeVilbiss Reusable Nebulizer is dishwasher safe, but do not place any DeVilbiss Disposable Nebulizer parts in an automatic dishwasher; doing so could cause damage.

Disinfect Daily:

1. Using a clean container or bowl, soak items in three parts hot water to one part white vinegar for 30 minutes (Fig. 8) OR use a medical bacterial-germicidal disinfectant available through your provider. Be sure to follow manufacturer's instructions carefully.
2. With clean hands, remove items from disinfectant solution, rinse under hot tap water, and air dry on a clean paper towel. Store in a zip-lock bag.

NOTE– Do not towel dry nebulizer parts; this could cause contamination.

WARNING!

To prevent possible risk of infection from contaminated cleaning solutions, always prepare fresh solution for each cleaning cycle and discard solution after each use.

3. Keep the outer surface of the tubing dust-free by wiping regularly. Nebulizer tubing does not have to be washed because only filtered air passes through it.

NOTE– The DeVilbiss Disposable Nebulizer will last at least 15 days and possibly longer, depending on usage. Proper cleaning will also help extend the life of the nebulizer. Because it is disposable, we recommend that an extra nebulizer be kept on hand at all times. The DeVilbiss Reusable Nebulizer is dishwasher safe and may be cleaned and reused for up to one year.

COMPRESSOR CLEANING

1. With power switch in the “○” position, unplug power cord from wall outlet.
2. Wipe outside of the compressor cabinet with a clean damp cloth every few days to keep dust-free.

! DANGER!

Do not submerge in water; doing so will result in compressor damage.

FILTER CHANGE

1. Filter should be changed every 6 months or sooner if filter discolors.
2. Remove filter (Fig. A) by grasping it firmly and pulling out of the unit. Discard old filter.
3. Replace with a new DeVilbiss filter (part #3655D-601). Additional filters should be purchased from your DeVilbiss provider.
4. Push new filter into position.

! CAUTION!

Do not reuse or wash filter or substitute any other material such as cotton for a DeVilbiss air-inlet filter. It will result in compressor damage.

MAINTENANCE

All maintenance must be performed by a qualified DeVilbiss provider or authorized service center.

! DANGER!

Electric shock hazard. Do not remove compressor cabinet. All disassembly and maintenance must be done by a qualified DeVilbiss provider.

Size	7.5" W x 4.0" H x 6.8" D (19.1 cm x 10.2 cm x 17.3 cm)
Weight	4.1 lbs. (1.86 kg)
Electrical Requirements	
3655DX & 3655TX	2.5 amps, 115 VAC ($\pm 10\%$) 60 Hz
3655IX	1.4 amps, 230 VAC ($\pm 10\%$) 50 Hz
3655KX & 3655PX	1.4 amps, 230 VAC ($\pm 10\%$) 60 Hz
Sound Level	56 dBA
Power Consumption	
3655DX & 3655TX	140 watts maximum
3655IX, 3655KX & 3655PX	160 watts maximum
Maximum Compressor Pressure	35 psig or greater (241 kPa)*
Compressor Free Air Flow	9 lpm or greater*
Operating Temperature Range	+40° to +104°F (+5 to +40°C)
Operating Humidity	up to 95% non-condensing
Storage/Transport Temperature Range	-40° to +158°F (-40° to +70°C)
Storage/Transport Humidity	up to 95% non-condensing
Supplied Nebulizer Capacity	6 ml (cc)
Supplied Nebulizer Nebulization Rate	0.15 ml/min or greater*
Supplied Nebulizer MMAD	5 microns or less**

*Conditions may vary based on altitude above sea level, barometric pressure, and temperature.

**Value determined with minimum flow rate through the nebulizer (20°C, sea level).

There are no hazardous materials used in the construction of this unit, therefore, there are no known risks associated with disposal of the equipment.

This product (models 3655DX & 3655TX) complies with the following electromagnetic compatibility standard: IEC 60601-1-2:2nd Edition

3655DX & 3655TX Equipment Classifications:

Protection against electric shock: Class II

Degree of protection against electric shock:
Type BF

Equipment not suitable for use in the presence of flammable anaesthetic mixture with air or with oxygen or nitrous oxide.

Mode of operation: Continuous operation

FIVE-YEAR LIMITED WARRANTY ON COMPRESSOR PARTS

All DeVilbiss Compressor models are warranted to be free from defective workmanship and material for a period of five years from the date of purchase. Any defective part(s) will be repaired or replaced at DeVilbiss' option if the unit has not been tampered with or used improperly during that period. Make certain that any malfunction is not due to inadequate cleaning or failure to follow the instructions. If repair is necessary, contact your authorized DeVilbiss provider or DeVilbiss Service Department at 800-338-1988 (814-443-4881) (Canada 800-263-3390 or 905-660-2459) for instructions. NOTE— This warranty does not cover providing a loaner compressor, compensating for costs incurred in compressor rental while said unit is under repair, or costs for labor incurred in repairing or replacing defective part(s).

THERE IS NO OTHER EXPRESS WARRANTY. IMPLIED WARRANTIES, INCLUDING THOSE OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE EXPRESS LIMITED WARRANTY AND TO THE EXTENT PERMITTED BY LAW ANY AND ALL IMPLIED WARRANTIES ARE EXCLUDED. THIS IS THE EXCLUSIVE REMEDY AND LIABILITY FOR CONSEQUENTIAL AND INCIDENTAL DAMAGES UNDER ANY AND ALL WARRANTIES ARE EXCLUDED TO THE EXTENT EXCLUSION IS PERMITTED BY LAW. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, OR THE LIMITATION OR EXCLUSION OF CONSEQUENTIAL OR INCIDENTAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

MANUFACTURER'S NOTE

Thank you for choosing a DeVilbiss compressor/nebulizer. We want you to be a satisfied customer. If you have any questions or comments, please send them to our address on the back of this manual.

FOR SERVICE CALL YOUR AUTHORIZED DEVILBISS PROVIDER:

Phone _____
Purchase Date _____
Serial #/Manufacture Date _____

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©2007 DeVilbiss Healthcare 05.07
Sunrise Medical Canada, Inc.
Sunrise Medical Pty. Limited
of Sunrise Medical HHG, Inc.